HOW TO USE OPERA'S PUMPING MODES

Vacuum strength and cycle speeds can be adjusted manually and uniquely for either breast, enabling a personalized experience.

Recommended Settings

It is generally recommended to start in Massage Mode on Cycle 100 / Vacuum 1, and slowly increase the vacuum until you have reached a letdown and the milk has started flowing. Once this happens, switch to Expression Mode starting on Cycle 42 / Vacuum 4 or 5 (or whatever level you were using in Massage Mode).

Feel free to customize the vacuum on each side according to each breast's needs.

It is recommended to use the alternate mode when double pumping for a sooner let down.

It is recommended to use the Alternate mode during your full pumping session for those moms that need extra stimulation to increase their milk supply.

Changing Modes

Press this button to quickly change modes. Single Left, Single Right, Alternate, and Synchronous

The current pumping mode will be displayed on the top-right corner

Toggle between EXPRESSION and MASSAGE by pressing this button.

ADJUSTING BOTH SIDES AT THE SAME TIME

When Opera is in **SYNCHRONOUS** or **ALTERNATE** mode, adjusting the vacuum or cycle will increase or decrease both sides at the same time.

SYNCHRONOUS mode express both breasts at the same time.

ALTERNATE mode express both breasts in an alternate pattern.

SYNCHRONOUS MODE

Increasing or decreasing the vacuum level and/or cycle will affect both sides.

ALTERNATE MODE

Screen will show vacuum level and cycle for both sides.

ADJUSTING ONE SIDE ONLY

Opera's dual motors technology allows moms to adjust each side independently.

Press **MODE** to select the side you want to use. (Left or Right)

Once the side is selected, you can modify the vacuum level and cycle for that side.

SINGLE L MODE

Increasing or decreasing the vacuum level and/or cycle will affect the currently selected side only.

SINGLE R MODE

Screen will show the current vacuum level and cycle for the selected side.


USING DIFFERENT SUCTION LEVELS ON EACH SIDE WHILE DOUBLE PUMPING

Once you have adjusted each side to your preference, you can press **MODE** to select **SYNCHRONOUS** or **ALTERNATE** mode.

Opera will keep the vacuum level selected for each side. However, cycles will be the same for both sides.

Opera will use the lowest cycle setting for both sides.

Increasing or decreasing the vacuum level and/or cycle will affect both sides. But the difference in suction level between sides will remain

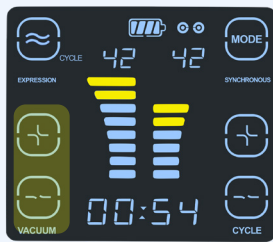Once one side reached max vacuum level, if you keep increasing the level it will only affect the side that had a lower level until both sides reach the maximum level.

The same will happen if you decrease the vacuum level. The difference in vacuum will remain until both reach the lowest level. (0)

SYNCHRONOUS OR ALTERNATE MODE

Want to learn more about Opera ?

SCAN ME

Escan the QR code to see instructional videos on how to use, assemble and sanitize your Opera pump.

Or visit: www.unimomus.com/pages/opera-videos

For more information visit www.Unimomus.com

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